microTargeting™ DP Stylets

Directions For Use

L011-81-01 (Rev A0, 2017-01-31)

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microTargeting™ DP Stylets Directions For Use

	WARNING / Caution, consult documents		Manufacturer
	Read usage instructions		Telephone number
	Not to be used in case package is damaged		Authorized Representative in the European Community
	Re-use is not allowed		European Conformity. This device fully complies with MDD Directive 93/42/EEC and legal responsibilities as a manufacturer are with FHC, Inc., 1201 Main Street, Bowdoin, ME 04287 USA.
	Do not resterilize		Temperature limitation: Storage conditions are between 0°C - 40°C (32°F - 104°F)
	Non-pyrogenic		Humidity limitation: Not to exceed 95%
	Catalog number		Sterilized using ethylene oxide
	Batch code		Non sterile
	Use by date		
	In reference to "Rx only" symbol; this applies to USA audiences only		

Rx only Caution- Federal law (USA) restricts this device to sale by or on the order of a physician.

Sterile DP Stylets

STERILE EO

- Sterile Medical Device – Do NOT resterilize.
- Do not use the contents if there is any evidence of damage to the package or package seal that could compromise sterility.
- For single patient use only
- Do not reuse; reusing single-use medical devices could lead to serious patient injury.

Non Sterile DP Stylets

NON STERILE

- If your microTargeting™ DP Stylets were shipped in a standard plastic box, DO NOT attempt to sterilize the stylets using this package. Remove microTargeting™ DP Stylets from the package prior to sterilization. FHC recommends use of the FHC sterilization tray (holds seven components).
- FHC has validated and recommends the following steam sterilization parameters:

Prevacuum wrapped (in 2 layers of 1-ply polypropylene wrap ⁽¹⁾) Preconditioning pulses: 3 Exposure time: 4 minutes at 132°C (270°F) Minimum dry time: 30 minutes	Gravity unwrapped Exposure time: 10 minutes at 132°C (270°F)
---	--
- ⁽¹⁾ Cycle was validated using Halyard Health H600 wrap
- FHC has validated and recommends the following manual cleaning protocol:
 - Immediately following use, thoroughly rinse each stylet under tap water.
 - Immerse all components in enzymatic wash solution for 1 minute. Remove from the wash solution, rinse with tap water, then distilled water.
 - Dry devices using a clean soft cloth.
- There is no restriction on the number of reuses providing user performs pre-use check.

The FHC microTargeting™ DP Stylets are nonpowered hand-held instruments intended to cut, hold or manipulate tissue during stereotactic functional neurosurgical procedures.

Safety Information

- Not intended for implantation

Contraindications

- microTargeting™ DP Stylets are not suited for chronic implantation.

WARNINGS

- Rx only: Caution-** Federal law (USA) restricts this device to sale by or on the order of a physician.
- In the event that a patient must be defibrillated via electric shock, the microTargeting™ DP Stylet should be withdrawn prior to defibrillation.
- To prevent electrical shock hazard DO NOT connect microTargeting™ DP Stylets to any line voltage source or any unknown power source.
- When there is a stylet in the brain, every effort should be made to minimize lateral forces to the drive/stereotactic system.

Directions For Use

- Adjust the stereotactic positioning system for the desired trajectory and depth.
- Insert the stylet into the stereotactic positioning system.
- Puncture the dura.
- After use, withdraw the stylet from the stereotactic positioning system.
- Refer to the directions for use associated with the insertion tubes and microelectrodes.
- Proceed with cleaning instructions for non sterile microTargeting™ DP Stylets, otherwise place the stylet in an approved sharps disposal container according to hospital protocol.

Pre-Use Check

WARNING: Delicate stylets should always be carefully inspected before use because damage to them can impact patient safety. Handle stylets carefully to prevent bending.

WARNING: Check to be certain that stylets are straight; with the collar over the edge, roll the microTargeting™ DP Stylet over a flat surface or in the measuring fixture slot.