## 28cm Lead Insertion Tubes

For use with stereotactic frame in an array configuration

### Directions For Use

L011-60 (Rev C0, 2017-08-18)

Contains directions for the following products:

66-IT-1028

[www.fh-co.com](http://www.fh-co.com)

**FHC, Inc.**  
1201 Main Street  
Bowdoin, ME 04287 USA  
Fax: +1-207-666-8292  
[www.fh-co.com](http://www.fh-co.com)

24 hour technical service:  
1-800-326-2905 (US & Can)  
+1-207-666-8190

**FHC Europe**  
(TERMOBIT PROD srl)  
42A Barbu Vacarescu Str, 3rd Fl  
Bucharest 020281 Sector 2  
Romania

**FHC Latin America**  
Calle 6 Sur Cra 43 A-200  
Edificio LUGO Oficina 1406  
Medellin-Colombia

## Contents

English	4
French - Français	7

## Indications for use:

Insertion Tubes are components of the microTargeting™ Drive and Star™ Drive Systems. The Drive Systems are intended to be used with commercially available stereotactic positioning systems for neurosurgical procedures which require the accurate positioning of microelectrodes, stimulating electrodes or other instruments in the brain or nervous system.

## Intended use:

The FHC microTargeting™ 28cm Lead Insertion Tube is intended for use by a neurosurgeon to accurately guide a microelectrode or instrument in the brain during stereotactic functional neurosurgical procedure.

## Contraindications:

microTargeting™ Insertion Tubes are not suited for chronic implantation. They have been validated for intracranial placement for 1 hour or less.

## Symbol Key:

	WARNING / Caution, consult documents		Manufacturer
	Read usage instructions		Telephone number
	Not to be used in case package is damaged		Authorized Representative in the European Community
	Re-use is not allowed		European Conformity. This device fully complies with MDD Directive 93/42/EEC and legal responsibilities as a manufacturer are with FHC, Inc., 1201 Main Street, Bowdoin, ME 04287 USA.
	Do not sterilize		Array configuration
	Non-pyrogenic		Temperature limitation: Storage conditions are between 0°C - 40°C (32°F - 104°F)
	Catalog number		Humidity limitation: Not to exceed 95%
	Batch code		Sterilized using ethylene oxide
	Use by date		In reference to "Rx only" symbol; this applies to USA audiences only
	<b>Rx only</b> Caution- Federal law (USA) restricts this device to sale by or on the order of a physician.		A patient with this device can not be scanned safely.

microTargeting™ and Star™ are trademarks of FHC, Inc.

## ! Safety Information:

STERILE EO

- For single patient use only
- Do not reuse; reusing single-use medical devices could lead to serious patient injury
- Not intended for implantation
- Sterile Medical Device – Do NOT sterilize
- Do not use the contents if there is any evidence of damage to the package or package seal that could compromise sterility

## ! Warnings and Cautions:

**USA** • **Rx only: Caution-** Federal law (USA) restricts this device to sale by or on the order of a physician.

- WARNING: Do NOT over-extend the lower tube as it could dislodge from the upper tube. Always use the stop tube and stylet to extend the lower tube.
- WARNING: All tubes used with the Star Drive or microTargeting Drive have been designed to be inserted and removed by hand. Any other tool should be used only as a last resort and indicates system repair may be necessary.
- WARNING: Delicate insertion tubes should always be carefully inspected before use because damage to them can cause targeting errors and impact patient safety. Handle insertion tubes and spacer tubes carefully to prevent bending.
- WARNING: The stylet should not be removed until tube is inserted in brain.
- WARNING: Check to be certain that tubes are straight; with the collar over the edge, roll the insertion tube over a flat surface or in the measuring fixture slot.
- CAUTION: Insertion tubes are not MRI compatible.
- WARNING: When there is an insertion tube in the brain, every effort should be made to minimize lateral forces to the Star Drive, microTargeting Drive or insertion tube, as it can translate into significant lateral movements of the tube in the brain.
- WARNING: Do not re-insert the stylet without the stop tube in place.
- WARNING: Do not rotate the lead while inserting it in the brain.
- WARNING: Do not apply torque between lower tube and the rest of the assembly.

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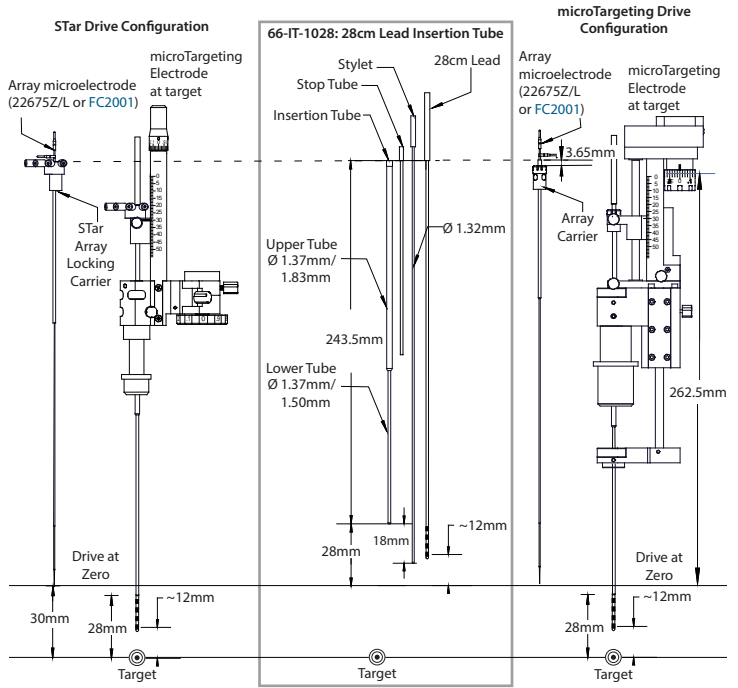
Ordering information: FHC / Medtronic

STar Drive Configuration

Items required for operation-  
STar Drive Manual Version: ST-DS-MA / FC8001  
STar Drive Motorized Version: ST-DS-ME / FC8002  
STar Array Locking Carrier: 70-AC-AR / FC8008  
28cm Lead (diameter should be adequate to fit  
inside a min 1.37mm ID tube)

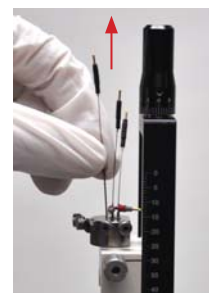
microTargeting Drive Configuration

Items required for operation-  
microTargeting Drive: MT-DS-01 / FC1006  
Array Carrier and Clamp: 66-AC-AR / FC1021  
28cm Lead (diameter should be adequate  
to fit inside a min 1.37mm ID tube)

Illustrative Procedure:

Directions for the 28cm Lead Insertion Tube are similar when used with a STar Drive or a microTargeting Drive. Dimensions remain the same, illustrations herein show the STar Drive.

After microelectrode recording has been performed and anatomical areas have been confirmed:

1. Remove all microelectrodes.

2. Remove insertion tube of selected track.

3. Insert lead insertion tube in selected track until fully seated. Holding the stylet handle down will ensure the stylet protrudes at the end of the insertion tube.

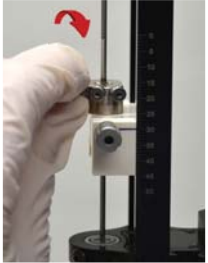
WARNING: The insertion tube will enter the brain at this stage.


WARNING: Never move the insertion tube in the brain without a stylet or electrode inside.

4. Remove all other microelectrode insertion tubes. (Remove lower guide spacer tubes if using a microTargeting Drive.)

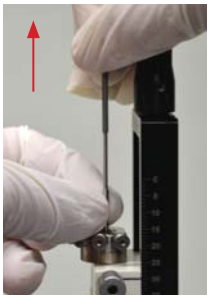5. Lock lead insertion tube in place.


6. Remove stylet while holding the stop tube collar down against the collar of the upper tube.

7. Remove stop tube.

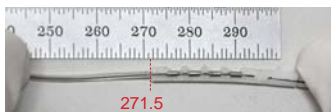Note: For additional support, microTargeting Drive users may raise the lower guide up until the upper tube engages in the corresponding hole of the lower guide bushing.

8. Align the bottom of the lead stylet handle with the 271.5mm graduation of a ruler.

9. Select lead contact to position at target. The distance between that contact and the end of the ruler represents the distance the drive needs to be advanced.

10. Insert 28cm lead in insertion tube until the lead stylet handle comes flush to the tube collar.

**!** WARNING: Insert the lead in one single motion.

**!** WARNING: Never attempt to retract the lead without first retracting the lower tube.

11. While holding the lead stylet handle, advance drive in order to position the selected lead contact (step 9) to target. Confirm lead location.

12. Carefully raise lower tube to expose the lead above the skull, while holding the lead stylet handle down.

13. Hold lead with smooth tip, rubber-coated tweezers. Remove the stylet from the lead and proceed with the removal of the drive system.

14. Secure the lead using method recommended by DBS manufacturer.

15. After use, the 28cm lead insertion tubes should be placed in an approved disposal container according to hospital protocol.

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microTargeting™ et STar™ sont des marques de commerce de FHC, Inc.