microTargeting™ DP Stylets

Directions For Use

L011-81-01 (Rev A0, 2017-01-31)

Contains directions for the following products:

66-ST-x (where x represents the following variations- Length: 15mm – 300mm, Outside diameter: 0.5-2.0mm, Collar size: 1-5mm diameter, "P" indicates presterile)

www.fh-co.com

FHC, Inc.
1201 Main Street
Bowdoin, ME 04287 USA
Fax: +1-207-666-8292
www.fh-co.com

EC REP

FHC Europe
(TERMOBIT PROD srl)
42A Barbu Vacarescu Str, 3rd Fl
Bucharest 020281 Sector 2
Romania

24 hour technical service:
1-800-326-2905 (US & Can)
+1-207-666-8190

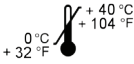
FHC Latin America
Calle 6 Sur Cra 43 A-200
Edificio LUGO Oficina 1406
Medellin-Colombia

Contents

English	4
French - Français	6

microTargeting™ DP Stylets Directions For Use

	WARNING / Caution, consult documents		Manufacturer
	Read usage instructions		Telephone number
	Not to be used in case package is damaged		Authorized Representative in the European Community
	Re-use is not allowed		European Conformity. This device fully complies with MDD Directive 93/42/EEC and legal responsibilities as a manufacturer are with FHC, Inc., 1201 Main Street, Bowdoin, ME 04287 USA.
	Do not re-sterilize		Temperature limitation: Storage conditions are between 0°C - 40°C (32°F - 104°F)
	Non-pyrogenic		Humidity limitation: Not to exceed 95%
	Catalog number		Sterilized using ethylene oxide
	Batch code		Non sterile
	Use by date		
	In reference to "Rx only" symbol; this applies to USA audiences only		

Rx only **Caution-** Federal law (USA) restricts this device to sale by or on the order of a physician.

Sterile DP Stylets

STERILE EO

-   • Sterile Medical Device – Do NOT re-sterilize.
-  • Do not use the contents if there is any evidence of damage to the package or package seal that could compromise sterility.
- For single patient use only
-  • Do not reuse; reusing single-use medical devices could lead to serious patient injury.

Non Sterile DP Stylets

NON STERILE

-  • If your microTargeting™ DP Stylets were shipped in a standard plastic box, DO NOT attempt to sterilize the stylets using this package. Remove microTargeting™ DP Stylets from the package prior to sterilization. FHC recommends use of the FHC sterilization tray (holds seven components).
- FHC has validated and recommends the following steam sterilization parameters:

<p>Prevacuum wrapped (in 2 layers of 1-ply polypropylene wrap ⁽¹⁾) Preconditioning pulses: 3 Exposure time: 4 minutes at 132°C (270°F) Minimum dry time: 30 minutes</p> <p>⁽¹⁾ Cycle was validated using Halyard Health H600 wrap</p>	<p>Gravity unwrapped Exposure time: 10 minutes at 132°C (270°F)</p>
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------
- FHC has validated and recommends the following manual cleaning protocol:
 1. Immediately following use, thoroughly rinse each stylet under tap water.
 2. Immerse all components in enzymatic wash solution for 1 minute. Remove from the wash solution, rinse with tap water, then distilled water.
 3. Dry devices using a clean soft cloth.
- There is no restriction on the number of reuses providing user performs pre-use check.

The FHC microTargeting™ DP Stylets are nonpowered hand-held instruments intended to cut, hold or manipulate tissue during stereotactic functional neurosurgical procedures.

Safety Information

- Not intended for implantation

Contraindications

- microTargeting™ DP Stylets are not suited for chronic implantation.

WARNINGS

- **Rx only: Caution-** Federal law (USA) restricts this device to sale by or on the order of a physician.
- In the event that a patient must be defibrillated via electric shock, the microTargeting™ DP Stylet should be withdrawn prior to defibrillation.
- To prevent electrical shock hazard DO NOT connect microTargeting™ DP Stylets to any line voltage source or any unknown power source.
- When there is a stylet in the brain, every effort should be made to minimize lateral forces to the drive/stereotactic system.

Directions For Use

1. Adjust the stereotactic positioning system for the desired trajectory and depth.
2. Insert the stylet into the stereotactic positioning system.
3. Puncture the dura.
4. After use, withdraw the stylet from the stereotactic positioning system.
5. Refer to the directions for use associated with the insertion tubes and microelectrodes.
6. Proceed with cleaning instructions for non sterile microTargeting™ DP Stylets, otherwise place the stylet in an approved sharps disposal container according to hospital protocol.

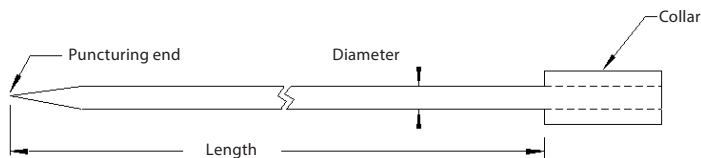
Pre-Use Check

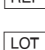
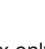
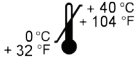
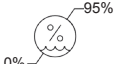
WARNING: Delicate stylets should always be carefully inspected before use because damage to them can impact patient safety. Handle stylets carefully to prevent bending.

WARNING: Check to be certain that stylets are straight; with the collar over the edge, roll the microTargeting™ DP Stylet over a flat surface or in the measuring fixture slot.

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